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# Particle filtering half mask – Instruction for use

Particle filtering half mask single use compliant with PPE Regulation (EU) 2016/425 and EN 149:2001+A1:2009

## Indications for use:

The particle filtering half mask is intended for use to protect against solid and liquid aerosols. It is intended for single use.

## Limitations:

Not suitable for use in workplaces with open flames (e.g. welding, casting) and for use in the presence of oxygen deficiency and toxic gases.

## Checks prior to use:

Please check whether the packaging is damaged, whether the mask body is damaged, deformed or has other obvious defects before use. Do not use damaged masks.

## Precautions for use:

- Please check the proper fit of the mask before use.
- It cannot be guaranteed that the requirement for tightness will be met if there is a lot of facial hair.
- Do not use the mask when air quality is poor, such as when the air is highly contaminated, or oxygen is depleted
- Do not use the mask in explosive environments.
- Dispose of the mask after use according to local regulations.
- The mask is intended for single use. Do not use the mask more than once.

## Fitting:

- Hold the ear loops with both hands and place the mask on your face so that your nose, mouth and chin are covered.
- Place the ear loops behind your ears.
- Adjust the ear loops with the plastic hook so that the mask fits your face.
- Adjust the nose clip to the shape of your nose so that the mask fits snugly.

## Storage condition:

The product should be stored in a well ventilated, dark and dry environment. Keep away from fire, pollutants and possible pollution sources.

Storage temperature: -20°C – 40°C; relative humidity: < 80%

## Shelf life:

The particle filtering half mask has a shelf life of 3 years from the date of production under the above storage conditions.

## EXPLANATION OF THE STANDARDS AND PICTOGRAMS

	Manufacturer – name and address		for single-use only and non-reusable		Lot number / batch code
	Shelf life		Read the instructions for use carefully before use.		Temperature delimitation
	Protect from moisture and maximum relative humidity of storage conditions		Protect from sunlight		This is a latex free product.